

NEW

## GraftMax™ Button

Adjustable Cortical Fixation Device



Keep your expectations fixed,  
**but your button**  
adjustable


MAXIMUM FIXATION + MAXIMUM GRAFT FILL + MAXIMUM REPRODUCIBILITY

# ADJUSTABLE LOOP FIXATION MEETS FIXED LOOP GRAFT STRENGTH

## + MAXIMUM FIXATION

You want the benefits of adjustable loop fixation with the graft fixation of a fixed loop. Now data shows you can have both. GraftMax buttons provide graft fixation that's equivalent to fixed loop devices with **70% less displacement** than other adjustable buttons.\*

### Creep analysis of cyclic loading at 250N over 1000 cycles

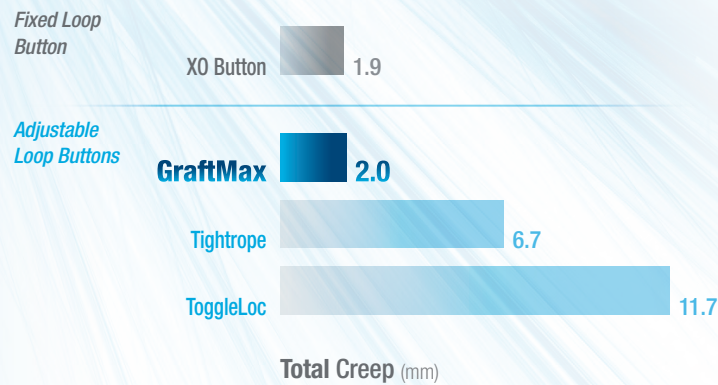

## DUAL-FIXATION MECHANISM REDUCES DISPLACEMENT


Other adjustable buttons feature purely friction-based fixation. GraftMax buttons have unique dual mechanical and friction fixation that uses cyclic force to compress suture into the locking pocket. This provides the strong data above.



Mechanical locking pocket

## + MAXIMUM GRAFT FILL

With other adjustable buttons, you're limited to an adjustable loop length of 15mm. Now, you can get maximum graft in the tunnel – even in shorter tunnels – with GraftMax buttons that are adjustable to less than 5mm.

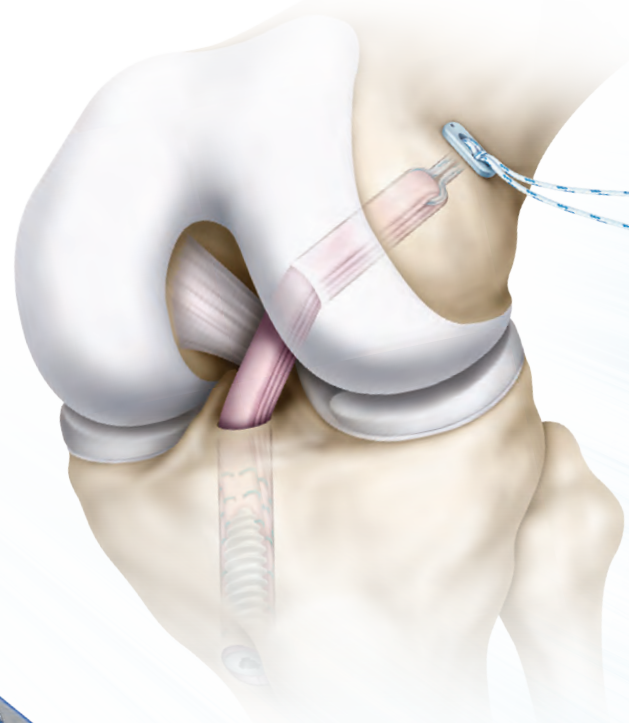

## + MAXIMUM REPRODUCIBILITY

Whether you use soft tissue, BTB or other closed loop grafts, GraftMax buttons help simplify your technique:

### Using a fixed loop?


GraftMax buttons are easier to insert and eliminate tedious calculations.

### Using another adjustable loop?

GraftMax buttons provide simpler and quicker graft advancement and tactile deployment confirmation.

## ANATOMIC TUNNELS MADE EASY

The NEW Curved Reaming System delivers the precise, anatomic tunnels you need – even helping you estimate the tunnel length before you drill.

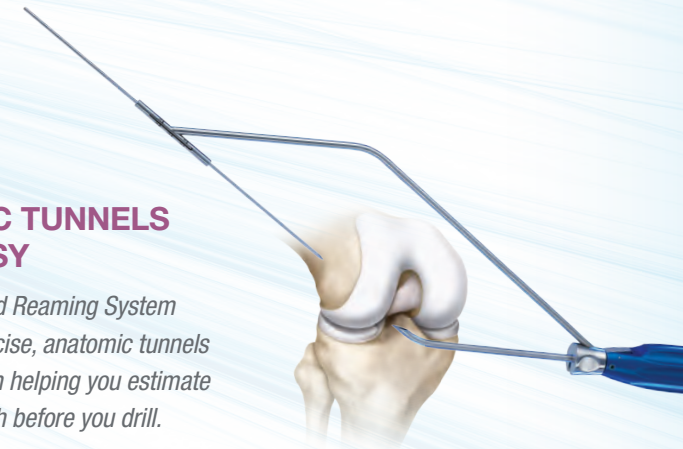

## CORTICAL BLOWOUT?

GraftMax buttons have you covered with a cradle that extends the button's use for tunnels up to 12mm.



GraftMax™ Button  
Adjustable Cortical Fixation Device

To schedule a demonstration, contact your local ConMed Sales Representative or visit [KneePreservationSystem.com](http://KneePreservationSystem.com).



Advancing the Future of Minimally Invasive Surgery. Together.

\*Data on file – TR14-464, 12/12/14

Knee Preservation System™

## GraftMax™ Button

Adjustable Cortical Fixation Device

To schedule a demonstration, contact your local  
ConMed Sales Representative or visit

[KneePreservationSystem.com](http://KneePreservationSystem.com)



525 French Road  
Utica, NY 13502

Local 727-392-6464  
Toll Free 1-800-237-0169

ConMed.com  
customer\_service@conmed.com